

MEDesign® Bedglasses

Product Reference BG




Description



The MEDesign® Bedglasses are optical prism spectacles which turn the world through 90 degrees. They are especially useful when a back pain sufferer is confined to bed since the user can watch television, read books or watch the world go by without needing to lift their head.

Use

Place the bedglasses on the head just like a standard pair of spectacles and look straight ahead. They can be worn over a normal pair of spectacles if required.

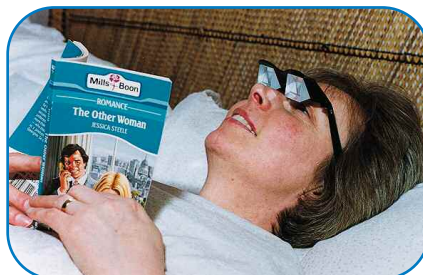

Specification

Prism: Manufactured to optical standards
Weight: 65g (2oz)

Care & Cleaning

Treat the Bedglasses like a normal pair of spectacles. Wash in warm water with a mild detergent, rinse well and dry thoroughly. The prism lenses can be cleaned with a standard spectacle lens cleaning cloth. Do not soak in disinfecting media, do not Autoclave, do not irradiate the Bedglasses.

©, MEDesign Ltd, 2001

Issue A12

