


MEDesign® Backfriend® Fleece Covers

Fitting Instructions For Backfriend Real Wool Fleece Covers

Backfriend Back Fleece Cover:

(Product Code/MPN: FCB - EAN: 5060464550928)


Fold Flap A on back panel around the bottom edge of the back rest and inside the fleece cover. Secure in place by using the fitted Velcro strips.

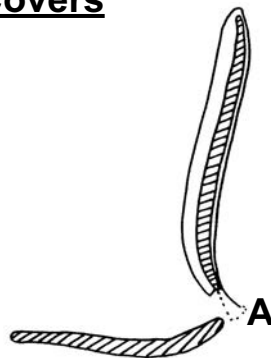

Backfriend Seat Fleece Cover:

(Product Code/MPN: FCS - EAN: 5060464550935)

Fold Flap B on fleece around the back edge of the seat base and over the back panel. Secure in place by using the fitted Velcro strips.

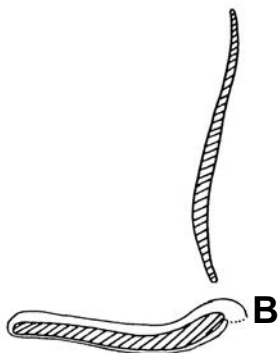

Washing Instructions:

Your real wool slipover fleece cover may be machine washed on the wool cycle at 30°C (86°F) and tumble dried at the cool setting. Before washing ensure that the two Velcro strips are firmly pressed together. Do not iron.

©, MEDesign Ltd, 1982-2018

Issue: F28



Backfriend



Backfriend Carrycase



Bedboards



Ortho Neck Pillows



Therapearl Wraps



Walking Sticks