


## MEDesign® Backfriend Fleece Covers

Product Reference FCB & FCS

### Fitting Instructions For Real Wool Fleece Covers


#### **Back Rest** (Product Code FCB):

Fold Flap A on back panel around the bottom edge of the back rest and inside the fleece cover. Secure in place by using the fitted Velcro strips.

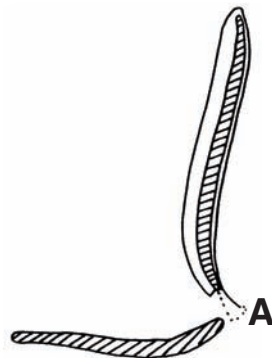

#### **Seat Base** (Product Code FCS):

Fold Flap B on fleece around the back edge of the seat base and over the back panel. Secure in place by using the fitted Velcro strips.

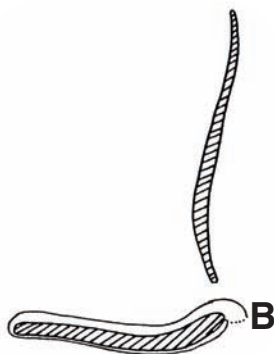

#### **Washing Instructions:**

Your real wool slipover fleece cover may be machine washed at 40°C (105°F) and tumble dried at the cool setting. Before washing ensure that the two Velcro strips are firmly pressed together. Do not iron.

©, MEDesign Ltd, 2010

Issue C20

