

## MEDesign® Prism Bed Glasses



### Description

The MEDesign® Prism Bed Glasses are optical prism spectacles which turn the world through 90 degrees. They are especially useful when a back pain sufferer is confined to bed since the user can watch television, read books or watch the world go by without needing to lift their head. They have also been used in MRI scanners to alleviate the effects of claustrophobia.

1 Always check suitability/compatibility before use



### Use

Place the Bed Glasses on the head just like a standard pair of spectacles and look straight ahead. They can be worn over a normal pair of spectacles if required.

### Specification

Prism: Manufactured to optical standards  
Weight: 65g (2oz)



### Care & Cleaning

Please note: MEDesign's Prism Bed Glasses are not a toy and are intended to be used by responsible adults. Treat the Bed Glasses like a normal pair of spectacles. Wash gently in warm water with a mild detergent, rinse well and dry thoroughly. The prism lenses can be cleaned with a standard spectacle lens cleaning cloth. Do not soak in disinfecting media, do not Autoclave, do not irradiate the Bed Glasses. Inspect the glasses before each use to ensure they are whole, undamaged and safe to use.

©, MEDesign Ltd, 1982-2018

Issue: F28



Backfriend



Raised Toilet Seats



Bedboards



Ortho Pillows



Wheat Hot Bags



Wedge Cushions